


Video Diagnostic System

User's Manual

1. Installation and Launch

- (1) Run VDSsetup to completion
- (2) Run VDS.exe

As shown below:

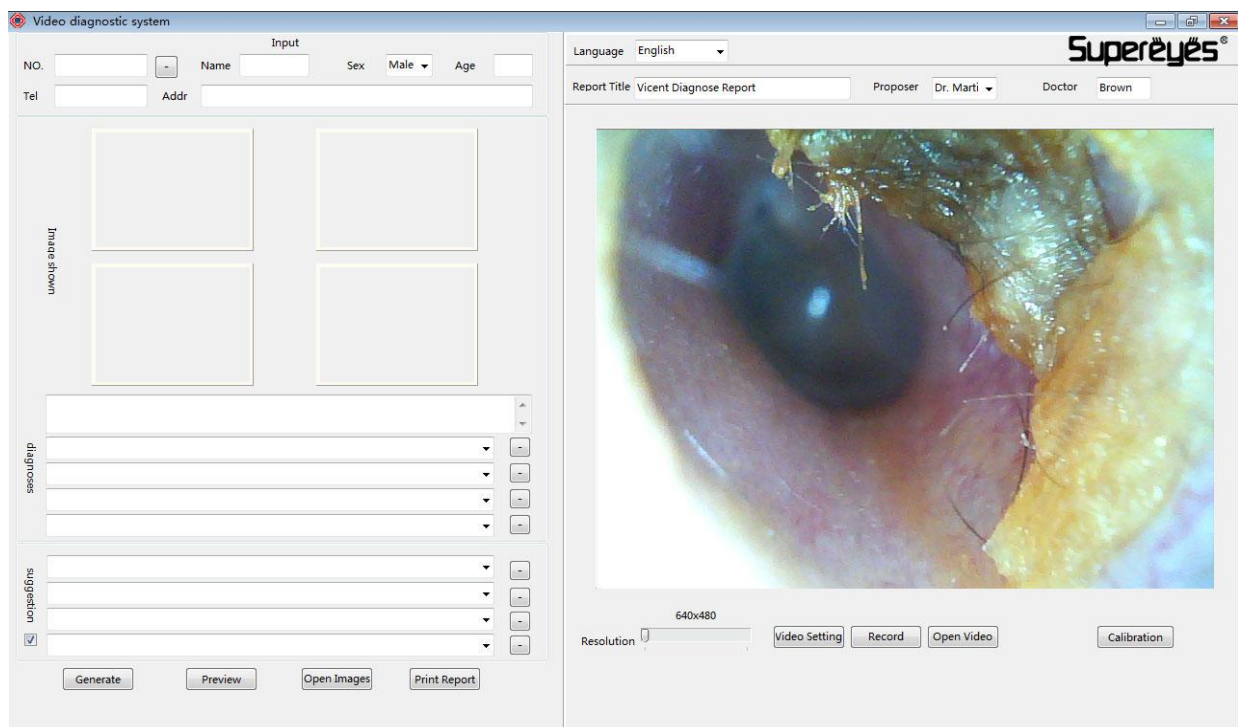

2. Add or Delete Information


(1) Add Information

In the specified field, enter the patient's name (required), age, gender, phone number, address and so on.

(The codes are numerical and auto-generated. When you enter an information that already exists, the system automatically displays the corresponding information)

Enter the report's title, the name of the applicant doctor (if a field is left blank, the information will not appear in the report) and the name of the diagnostic doctor.

(2) Delete Information

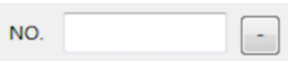Clicking "-"  on the right-hand side of the code will delete all


information for the current patient.

The screenshot shows the 'Video diagnostic system' window. On the left, there is an 'Input' section with fields for 'NO.' (1256), 'Name' (Zenith), 'Sex' (Female), 'Age' (28), 'Tel' (123456789), and 'Addr' (123 Rainbow Blvd, Utah, USA, 123456). Below this is a section labeled 'Image shown' with four empty thumbnail boxes. Further down are sections for 'diagnoses' and 'suggestion', each with a list of dropdown menus and a 'Generate' button. At the bottom left are buttons for 'Generate', 'Preview', 'Open Images', and 'Print Report'. On the right, there is a 'Language' dropdown set to 'English'. Below it is a 'Report Title' field (Vicent Diagnose Report) and fields for 'Proposer' (Dr. Marti) and 'Doctor' (Brown). A large video preview window shows a close-up of a human eye. Below the video is a 'Resolution' dropdown set to '640x480' and buttons for 'Video Setting', 'Record', 'Open Video', and 'Calibration'. The 'SuperEyes' logo is in the top right corner.

3. Observation and Take Photo

Click the left mouse button in the preview box to lock the screen in place

Double click the left mouse button in the preview box and the screen will be stored in a thumbnail box on the left-hand side. If you are unhappy with the image, you can click to select a thumbnail box, click the right mouse button to delete, or double-click the preview box to replace the image.

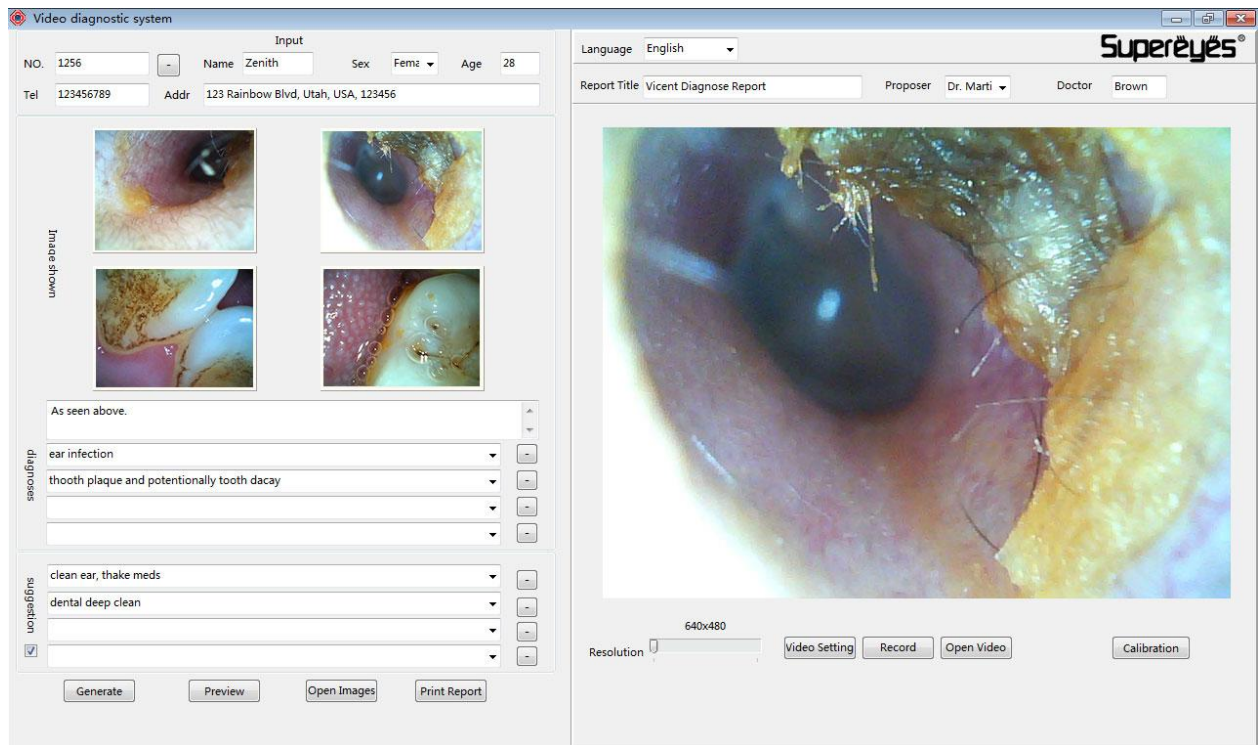



4. Video Setting

(1) Changing the resolution (on devices of 300 thousand or more resolution)



When the resolution is more than 300 thousand pixels, you can use the scale bar to change the resolution and magnify the image.

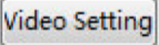
(2) Click  to bring up

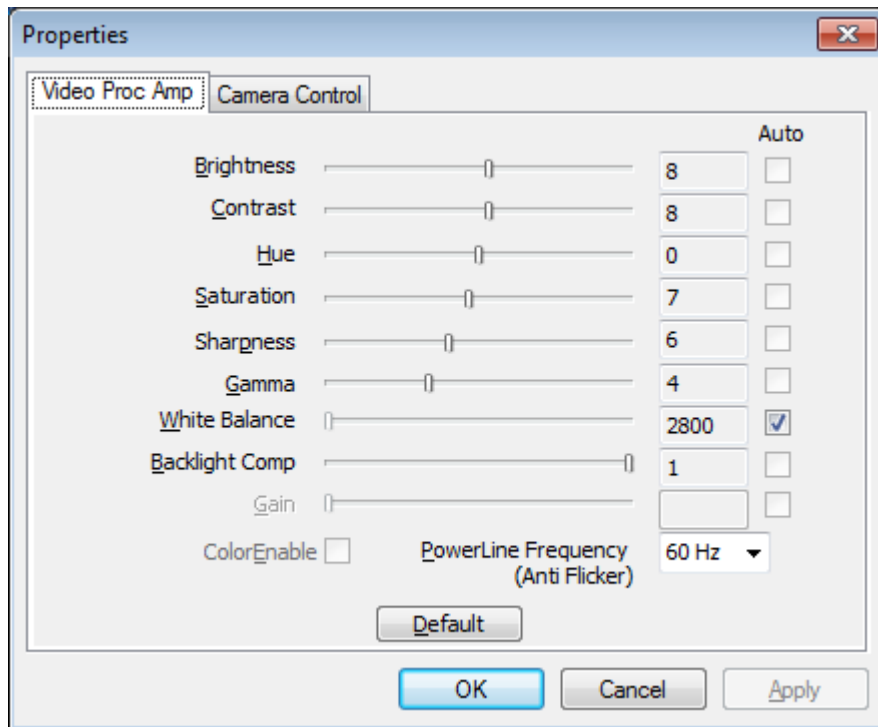

You can adjust the corresponding indicator. Usually, the default value gives the best outcome.

(3) Measurement Calibration

Put the standard scale in a clear position. Click **Calibration**. Move the mouse to the preview box until the mouse displays a small cross. Click the left mouse button, move it to an appropriate length and release. The following field should pop up:

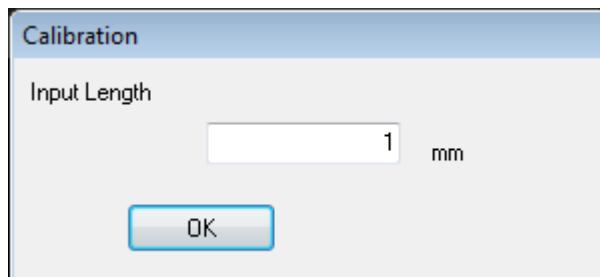

Enter the value to confirm and complete the calibration process.
There is no need to recalibrate if the focus is not readjusted.

5. Image Management

Double-click the thumbnail image. The image will show up in the preview box. You can manage the color, rotation, and measurement. Double-click to restore to its original location. Or click "Back" to cancel the operation.

Gray

Measure

Back

6. Medical History Notes

Type the corresponding content in the blank section

7. Diagnostic Options

Enter the opinion of the diagnostic doctor and save the generated report. For next time, click the triangle at the right-hand corner display the selection. If this is not required, click "-" on the right-hand side to delete.

As seen above.

diagnoses

ear infection

thooth plaque and potentially tooth dacay

8. Treatment Advice

Enter the opinion of the diagnostic doctor and save the generated report. For next time, click the triangle at the right-hand corner to display the selection. If this is not required, click "-" on the right-hand side to delete.

If the report does not require a doctor's treatment advice, just leave unchecked the box below treatment advice.

suggestion

clean ear, thake meds

dental deep clean


☒

9. Generate Report

Click "Generate" and save the content.

10. Preview Report

Click "Preview". The other page will display the current report. You can go forward, go backward, print or perform other operations.

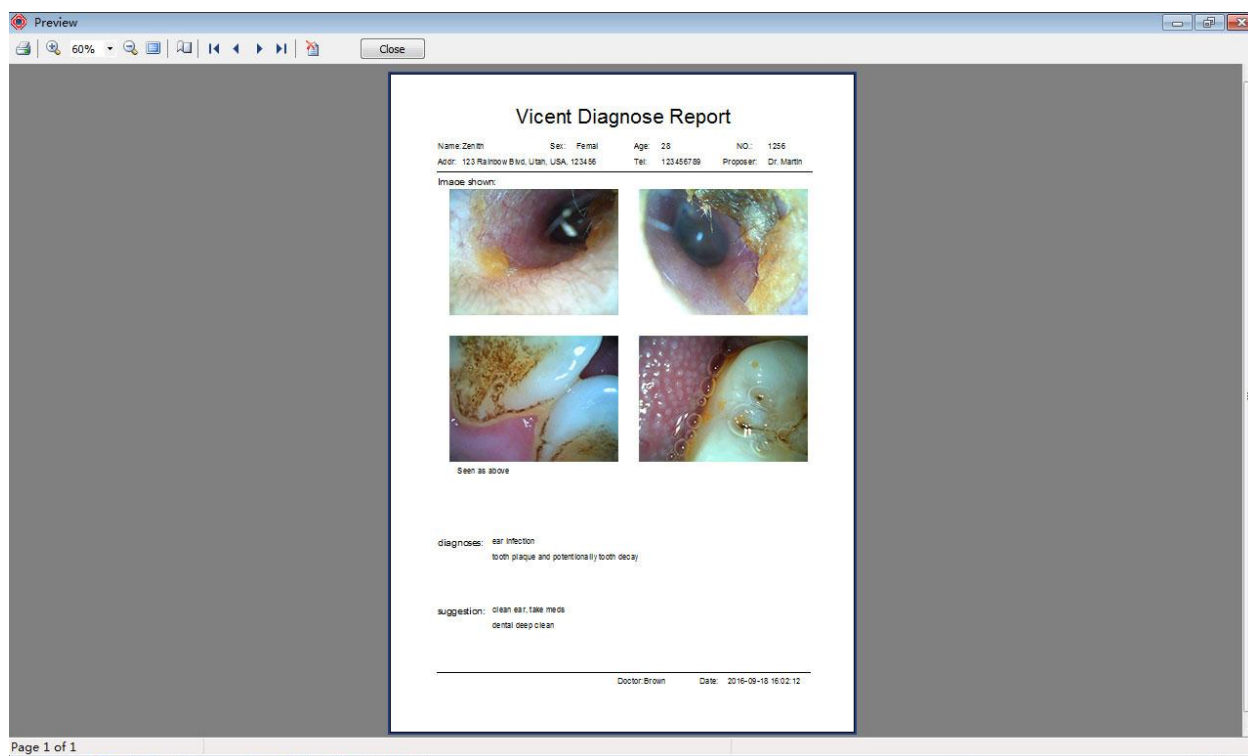


11. Print Report

Print reports directly.

Perform corresponding printing operation.

12. View Image

Click "Open Images" to enter current image directory.

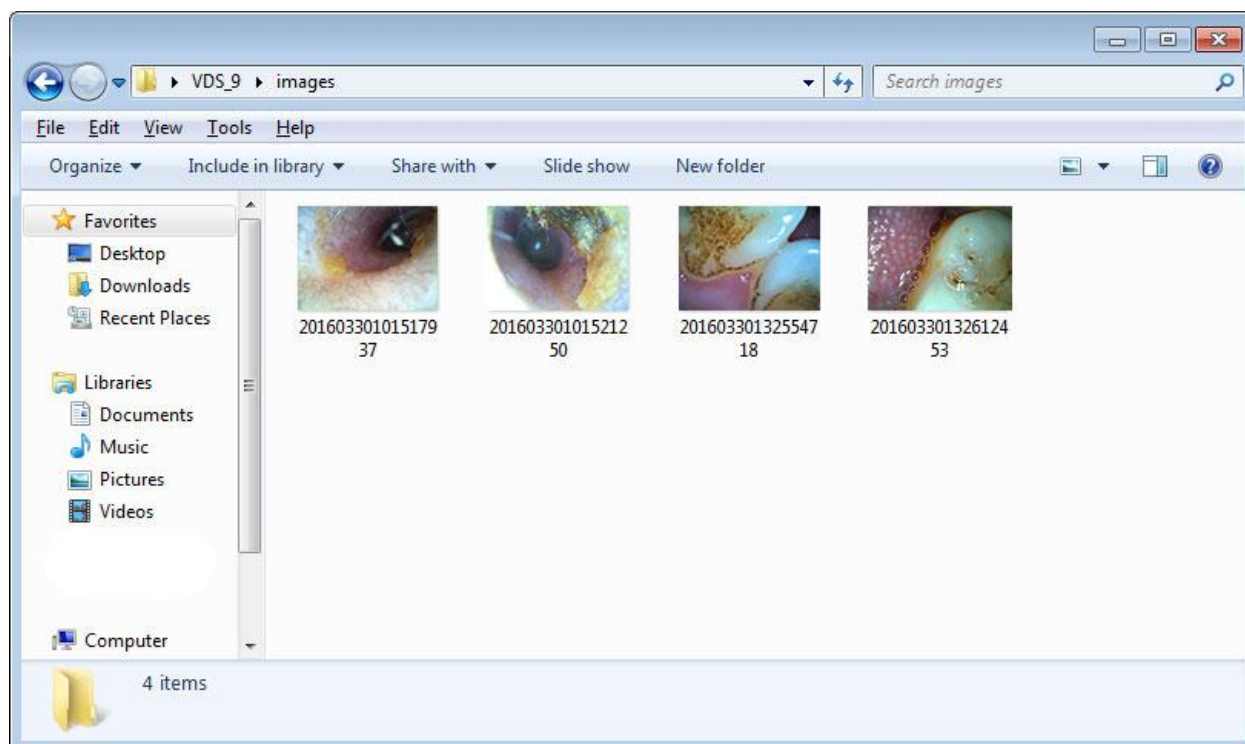

13. Video

Click the "Record" button to start or stop a video. Click the "Open" button to enter the current directory of the recorded video.



14. Help

Click the  link at the top right-hand corner to reach the service website.